

breCeram

From shaping to a high polish – finely tuned processing set, for the ceramics specialist



- inverted cone with relief grinding technology for smooth surfaces
- fine but abrasive diamond grinder
- two different abrasion stages of the Abraso-Fix-Roundbrushes permit rapid polishing, as the polish paste is already carried in the bristles
- Ceragum coarse is suitable for use with ceramic and for rubberized metal
- Cerafine very quickly gives ceramic and metal a high gloss



The diamond grinder is for the coarse processing of ceramic. But nevertheless a fine surface is achieved due to the fine diamonds.



The inverted cone is ideally suited to shaping of the mastication surfaces. Simultaneously the undercut produces a polished ceramic surface.



Ceragum coarse is a universal product. It removes material fast, leaving an optimal surface structure.



Abraso-Fix green is used to produce the coarse surface structure. It is also highly suitable for smoothing ceramic and metal mastication surfaces.




Abraso-Fix red already achieves a light polish on the surface. It is used to give the first polish to ceramic and metal.



Cerafine is the high gloss polisher for ceramic and metal. It is particularly suitable for transition areas of metal to ceramic, as after gloss burning the metal polish no longer has a matting effect.



Fast and simple processing of ceramic and metal. breCeram offers the ideal combination.

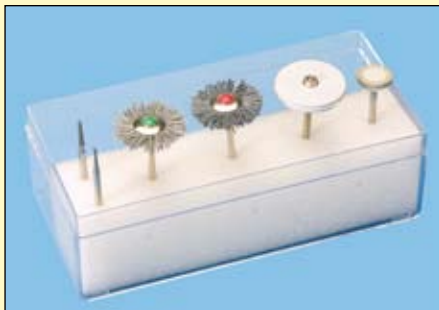

breCeram processing set for ceramics

- 6 pieces
 1 diamond grinder fine
 1 Tungsten carbide 1.2
 1 Abraso-Fix green
 1 Abraso-Fix red
 1 Ceragum coarse, wheel
 1 Cerafine, wheel
 Order No. 520 2028 6



Diamond grinder fine

Order No. 340 0107 1
1 piece



Tungsten carbide

Order No. H010 NH 12
1 piece



Abraso-Fix green

Order No. 350 0059 0, 2 pieces
Order No. 350 0075 5, 8 pieces



Abraso-Fix red

Order No. 350 0060 0, 2 pieces
Order No. 350 0075 3, 8 pieces



Ceragum coarse, wheel not mounted

Order No. PRK G221 2, 12 pieces
Order No. PRK G225 0, 50 pieces
Order No. PRK G225 0, 100 pieces



Cerafine, wheel

Order No. 520 2028 5
1 piece

Accessories:



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
Tungsten carbide

Order No. H010 NH 08
1 piece



Tungsten carbide

Order No. H010 NH 10
1 piece



Tungsten carbide

Order No. H010 NH 16
1 piece