

# crea.lign

powered by visio.lign

## Layering scheme for RED-WHITE individualization

### Layering instructions - crea.lign GUM



The bone is represented with beige.



The effect of depth is achieved with lila. Lila is spread from the mucolabial fold towards the teeth.



The layers that were previously applied are coated with pink.



The alveolar sockets are accentuated with rose.



Well perfused areas are accentuated with red.



Light is applied to the gingival margin.



The entire surface is sealed with transpa.

# crea.lign

powered by visio.lign

## Layering scheme for RED-WHITE individualization

### Layering instructions - crea.lign Paste GUM



Use Paste PO to contour the marginal area.



The alveolar sockets are built up with Paste PL.



Use light to apply the gingival margin.



The effect of depth is achieved with lila.



Well perfused areas are accentuated with red.



The entire surface is sealed with transpa.

